
SKU: UVCLED-TRI




## **UVC LED Dosimeters**

UVC LED Dosimeters are easy-to-use photochromic indicators that provide visible evidence of successful germicidal irradiation with UV-C devices emitting 260-280 nm wavelengths. #SeeYourSuccess

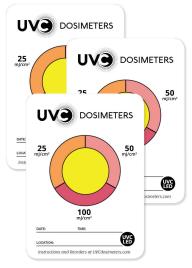

UVCLED-TRI
\*Also available in 1" Dot and 2"x2" Quick Check.

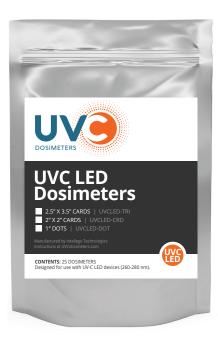

## Instructions for Use

- Remove adhesive backing and the UVC LED Dosimeter on or adjacent to any surface or equipment that will be disinfected with UV-C. Face yellow indicator toward light source and begin the disinfection cycle.
- After cycle is complete, make exposure readings immediately or within 24 hours for optimal results.
- For record-keeping, make notations directly on the dosimeter to indicate results. Retain or discard exposed dosimeters in accordance with site requirements.

- ✓ Designed for use with UVC-LED devices (260-280 nm)
- Available in three styles: TRI Card (2.5"x3.5")
  Quick Check (2"x2")
  Dot (1")
- Manufactured in Sweden and validated by Research Institute of Sweden (RISE)
- 25 units per bag
- Resealable mylar packaging (4.5" x 6.375")
- MOQ 200 units
- Single use only
- 3 year shelf life
- Private label and customization available

To order or request a quote, visit UVCdosimeters.com or email info@intellego-technologies.com



Online: UVCdosimeters.com
Online: Intellego-Technologies.com
Email: info@intellego-technologies.com